

QuikClot® COMBAT GAUZE

QuikClot® COMBAT GAUZE®



**Vacuum Packed and Z-Folded
For Temporary External Use
To Control Traumatic Bleeding**

ONE (1) 3 IN X 4 YDS (7.5 CM X 3.7 M) HEMOSTATIC DRESSING



Made in the USA

NSN 6510-01-562-3325

QuikClot® COMBAT GAUZE®

DIRECTIONS FOR USE



1. Open package and remove **Combat Gauze**. Keep the empty package.



2. Pack **Combat Gauze** into wound and use it to apply pressure directly over bleeding source. More than one **Combat Gauze** may be required.



3. Continue to apply pressure for 3 minutes or until bleeding stops.



4. Wrap and tie bandage to maintain pressure. Seek medical care immediately. Show **PRODUCT REMOVAL** directions on package to medical personnel.

CE
2797

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP
The Hague
The Netherlands

REF 200

PRODUCT REMOVAL: 1. Gently remove gauze from wound. 2. Thoroughly irrigate wound.

Manufactured by:
Z-Medica®, LLC
4 Fairfield Boulevard
Wallingford, CT 06492 USA
Tel: 1.877.750.0504, +1.203.294.0000
Fax: 1.800.343.8656, +1.203.303.7216
QuikClot.com
Patents: Z-Medica.com/patents.aspx
© 2020 Z-Medica, LLC
All rights reserved.
Contains kaolin.
100518-05-WEB Rev D 10/8/2020

FOR USE BY TRAINED EMERGENCY RESPONDERS



Do Not Reuse



WARNINGS: For external use only. Sterility not guaranteed if package is damaged or opened. Discard package if damaged. Avoid contact with eyes. Reuse will cause risk of infection and loss of efficacy.



STERILE R

MD Medical Device